



IntelliKeys USB Setup for Use as a Switch Interface with SoftTouch Software


1. Install the IntelliKeys USB Software for your computer.

2.. On Windows, you will typically see a 'USB' icon on your taskbar to the right side by the clock. If connected, the icon will be blue. If not connected, the icon will be red. Click on the icon and select 'Show IntelliKeys USB Control Panel'.

3. On Macintosh, from the Apple Menu, select Control Panels->Intellikeys USB



IntelliKeys USB Icon

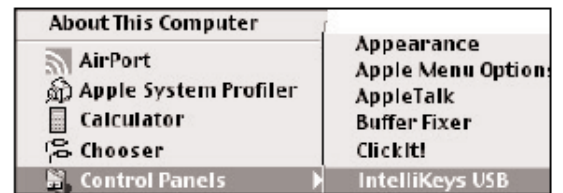

Apple menu listings

IntelliKeys USB Control Panel

4. Once the IntelliKeys USB Control Panel is open, click on the 'Advanced' Tab.

5. Next to 'As a switch, with these settings:', select '9 - 1 and 2' from the drop down list.

6. Click 'Done'

